

## PAS/Glycogen, For Digestion Tissue Control Slides

<b>Description:</b>	Human. Natural disease state (where applicable). Paraffin embedded.
<b>Availability:</b>	TCS0024-5     5 unstained, undigested slides. TCS0024-25   25 unstained, undigested slides.
<b>Storage:</b>	2-25° C.
<b>Suggested Stain Kit:</b>	PAD-1
<b>Procedure:</b>	See below.


### Periodic Acid Schiff (PAS) Diastase Stain Kit

**Description:** The Periodic Acid Schiff (PAS) Diastase Stain Kit is intended for use in histological demonstration of lymphocytes and mucopolysaccharides. The  $\alpha$ -Amylase digestion step acts on glycogen to break it into smaller sugars that are then washed off the tissue section allowing visual comparison of digested and undigested slides. The PAS reaction in tissue sections is useful for the demonstration of mucopolysaccharides.

PAS Positive Material:                      Magenta  
Nuclei:    Blue

**Uses/Limitations:** Not to be taken internally.  
For In-Vitro Diagnostic use only.  
Histological applications.  
Do not use if reagents become cloudy.  
Do not use past expiration date.  
Use caution when handling reagents.  
Non-Sterile.


**Control Tissue:** Liver

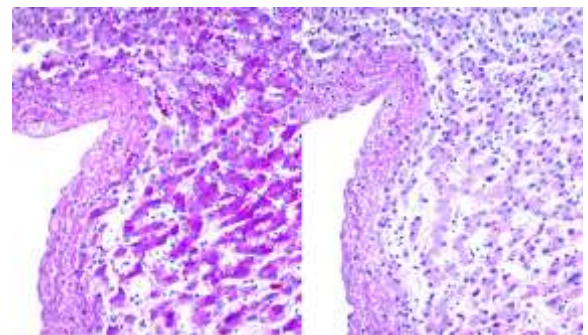


No Digestion

With Digestion


**Ordering information regarding individual components on back page!**

Storage: 2° C  25° C

Doc: IFU-TemplateMixedStorageev2

 ScyTek Laboratories, Inc.  
205 South 600 West  
Logan, UT 84321  
U.S.A.

  
Emergo Europe  
Prinsessegracht 20  
2514 AP The Hague, The Netherlands

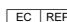**Kit Contents:**

<u>Item #</u>	<u>Kit Contents</u>	<u>Volume</u>	<u>Storage</u>
AAS250	Alpha-Amylase Solution (1%)	250 ml	2-8° C
PAQ250	Periodic Acid Solution	250 ml	2-8° C
SRF250	Schiff's Solution	250 ml	2-8° C
HMM125	Hematoxylin, Mayer's	2x125 ml	18-25°C
BRT125	Bluing Reagent	2x125ml	18-25°C

**Precautions:**

Avoid contact with skin and eyes.  
 Harmful if swallowed.  
 Follow all Federal, State, and local regulations regarding disposal.


**Procedure:**

1. Deparaffinize two identical sections if necessary and hydrate to distilled water.
  2. If sections are Zenker-fixed, remove mercuric chloride crystals using iodine and clear with sodium thiosulfate. Rinse in running tap water.
  3. Apply Alpha-Amylase Solution (1%) to one slide and incubate for 10-30 minutes at room temperature.
  4. Rinse in 2 changes of distilled water.
- Note: The remainder of this procedure is performed on both the “digested” and “undigested” slides.
5. Apply Periodic Acid Solution (1%) to tissue section and incubate for 5 minutes.
  6. Rinse slide in 4 changes of distilled water.
  7. Apply Schiff's Solution to tissue section and incubate for 10-20 minutes.
  8. Rinse slide in warm running tap water for 2 minutes.
  9. Rinse slide in distilled water.
  10. Apply Hematoxylin, Mayer's (Lillie's Modification) to tissue section and incubate for 1 minute.
  11. Rinse in running tap water for 1 minute followed by 2 changes of distilled water.
  12. Apply Bluing Reagent for 5 seconds and rinse in distilled water.
  13. Dehydrate through graded alcohols.
  14. Clear, and mount in synthetic resin.


**References:**

1. Culling CFA, Allison RT, Barr WT.: Cellular Pathology Technique, 4<sup>th</sup> Edition. Butterworths, Pages 216-220, 1985.
2. Sheenan, D.C., Hrapchak, B.B. Theory and Practice of Histotechnology, 2<sup>nd</sup> Edition. CV Mosby, Columbus, OH. Pages 164-167, 1980.


**Bulk Reagent Ordering Information and Current Pricing at [www.scytek.com](http://www.scytek.com)**

 Storage: 2° C  25° C

Doc: IFU-TemplateMixedStorageev2



ScyTek Laboratories, Inc.  
 205 South 600 West  
 Logan, UT 84321  
 U.S.A.

Emergo Europe  
 Prinsessegracht 20  
 2514 AP The Hague, The Netherlands

# Instructions For Use TCS0024-IFU


Rev. Date: March 6, 2017

Revision: 1


Page 3 of 3

P.O. Box 3286 - Logan, Utah 84323, U.S.A. - Tel. (800) 729-8350 – Tel. (435) 755-9848 - Fax (435) 755-0015 - [www.scytek.com](http://www.scytek.com)

Description:	Catalog #	Volume
Alpha-Amylase Solution (1%)	AAS250	250 ml
	AAS500	500 ml
	AAS999	1000 ml
Periodic Acid Solution	PAQ250	250 ml
	PAQ500	500 ml
	PAQ999	1000 ml
Schiff's Solution	SRF250	250 ml
	SRF500	500 ml
	SRF999	1000 ml
Hematoxylin, Mayer's (Lillie's)	HMM500	500 ml
	HMM999	1000 ml
	HMM3800	1 Gallon
Bluing Reagent	BRT500	500 ml
	BRT999	1000 ml
	BRT3800	1 Gallon

Storage: 2° C  25° C

Doc: IFU-TemplateMixedStorageev2

 ScyTek Laboratories, Inc.  
205 South 600 West  
Logan, UT 84321  
U.S.A.



Emergo Europe  
Prinsessegracht 20  
2514 AP The Hague, The Netherlands