

Biebrich Scarlet Solution (1%)

Description: Biebrich Scarlet Solution (1%) is used in Lillie's modification of Masson's Trichrome.

Cytoplasm:	Pink
Muscle:	Red
Erythrocytes:	Bright Red
Fibrin:	Red
Myelinated Fibers:	Red

Uses/Limitations: Not to be taken internally.
For In-Vitro Diagnostic use only.
Do not use past expiration date.
Use caution when handling reagents.
Non-Sterile


Availability:	<u>Item #</u>	<u>Volume</u>
	BST500	500 ml
	BST999	1000 ml


Storage: Store at 18-25°C.

Precautions: Avoid contact with skin and eyes.
Harmful if swallowed.
Follow all Federal, State, and local regulations regarding disposal.

Procedure:

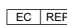
1. Deparaffinize sections if necessary and hydrate to distilled water.
2. Apply Biebrich Scarlet Solution (1%) and incubate for 3-5 minutes.
3. Rinse quickly in tap water followed by a quick rinse in distilled water.
4. Dehydrate quickly through graded alcohols.
5. Clear, and mount in synthetic resin.

Storage: 18° C  25° C



ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
U.S.A.

CE 

 EmergoEurope (31)(0) 70 345-8570
Molsnstraat 15
2513 BH Hague, The Netherlands