

Instructions For Use EYA-IFU		
Rev. Date: Jan. 26, 2008	Revision: 3	Page 1 of 2

P.O. Box 3286 - Logan, Utah 84323, U.S.A. - Tel. (800) 729-8350 - Fax (435) 755-0015 - www.scytek.com

Eosin Y Solution (Alcoholic)

Description: Eosin Y Solution is intended for use in the histological demonstration of cytoplasm and is commonly used as a counterstain for Hematoxylin. When used correctly, various shades of pink can be obtained to aid in visualization of tissue components. Erythrocytes, collagen, and the cytoplasm of muscle or epithelial cells will stain with different shades of pink.

Cytoplasm:	Pink to Red
Erythrocytes	Pink to Red
Nuclei:	Black/Blue (Hematoxylin)


Uses/Limitations: For In-Vitro Diagnostic use only.
Histological and Cytological applications.
Do not use past expiration date.
Use caution when handling these reagents.


Control Tissue: Any fixed or frozen tissue.
Cell smear.

Storage: Room Temperature (18-25° C.)

Availability/Contents:

<u>Item #</u>	<u>Volume</u>
EYA500	500 ml
EYA999	1000 ml
EYA010	10 Liters
EYA-20000	20 Liters
EYA-50000	50 Liters
EYA-100000	100 liters

Storage: 18° C  25° C

 ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
435-755-9848
U.S.A.

 Authorized Representative in Europe
 (Regulatory affairs only)
EmergoEurope (31)(0) 70 345-8570
Molsnstraat 15
2513 BH Hague, The Netherlands

Instructions For Use EYA-IFU

Rev. Date: Jan. 26, 2008

Revision: 3

Page 2 of 2

P.O. Box 3286 - Logan, Utah 84323, U.S.A. - Tel. (800) 729-8350 - Fax (435) 755-0015 - www.scytek.com

Precautions: Keep away from open flame.
Avoid contact with skin and eyes.
Harmful if swallowed.
Follow all Federal, State, and local regulations regarding disposal.
Use in chemical fume hood whenever possible.
Wear protective clothing.

Procedure (Standard):

1. Deparaffinize sections if necessary and hydrate to distilled water.
2. If sections are Zenker-fixed, remove mercuric chloride crystals using iodine and clear with sodium thiosulfate. Rinse in running tap water.
3. Rinse slide in distilled water.
4. Rinse slide in distilled water.
5. Stain slide in Hematoxylin, Mayer's (HMM125) for 5 minutes.
6. Rinse slide in running tap water for 2-3 minutes.
7. Apply Bluing Reagent (BRT125) for 30 seconds.
8. Rinse in distilled water.
9. Stain slide in Eosin Y Solution for 1-3 minutes.
10. Rinse quickly in 95% alcohol followed by 2 minutes in absolute alcohol.
11. Clear, and mount in synthetic resin.

References:

1. Sheenan, D.C., Hrapchak, B.B. Theory and Practice of Histotechnology, 2nd Edition. CV Mosby, Columbus, OH. Pages 140-141, 1980.
2. Lillie, R.D., Fullmer, H.M. Histopathologic Technique and Practical Histochemistry, 4th Edition, McGraw-Hill, NY. Pages 205-208, 1976.
3. Luna, L.G. Manual of Histologic Staining Methods of the Armed Forces Institute of Pathology, 3rd Edition, McGraw-Hill, NY, Pages 34-35, 1968.

Storage: 18° C



25° C



ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
435-755-9848
U.S.A.



Authorized Representative in Europe



(Regulatory affairs only)
EmergoEurope (31)(0) 70 345-8570
Molsnstraat 15
2513 BH Hague, The Netherlands