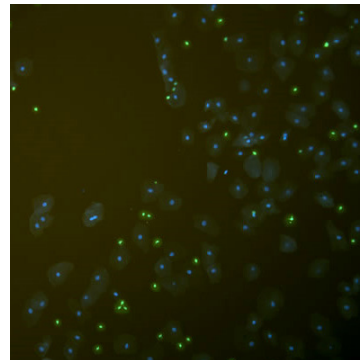


# FluoreGuard Mounting Medium (Hard Set)

**Description:** FluoreGuard Mounting Medium (Hard Set) is a water-soluble medium designed for semi-permanent coverslipping of fluorescent slide preparations. FluoreGuard dries to a semi-rigid layer which eliminates tissue damage due to moving coverslips and facilitates long term storage. The (Hard Set) formulation provides better drying in humid conditions.

**Uses/Limitations:** Not to be taken internally.  
For In-Vitro Diagnostic use.  
Histological applications.  
Do not use if reagents become cloudy.  
Do not use past expiration date.  
Use caution when handling reagents.  
Non-Sterile.



**Ordering Information and Current Pricing at [www.scytek.com](http://www.scytek.com)**


<b>Availability:</b>	<u>Item #</u>	<u>Volume</u>
	FMH030	30ml Dropper
	FMH060	60ml Dropper
	FMH500	500ml
	FMH999	1000ml


**Storage:** Store at 18-25°C.

**Precautions:** Avoid contact with skin and eyes.  
Harmful if swallowed.  
Follow all Federal, State, and local regulations regarding disposal.

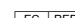
- Procedure:**
1. Apply several drops of FluoreGuard Mounting Medium (Hard Set) and coverslip.
  2. Allow FluoreGuard to dry for 30 minutes prior to viewing. Product will dry to a semi-solid form overnight.
  3. FluoreGuard is an aqueous based medium and may be removed by soaking in PBS overnight.

- References:**
1. Wanninger S, Donati M, Di Francesco A, Hässig M, Hoffmann K, Seth-Smith HM, Marti H, Borel N. Selective Pressure Promotes Tetracycline Resistance of Chlamydia Suis in Fattening Pigs. PloS one. 2016 Nov 28;11(11):e0166917.

Storage: 18° C  25° C

 ScyTek Laboratories, Inc.  
205 South 600 West  
Logan, UT 84321  
U.S.A.

  
Emergo Europe  
Molenstraat 15  
2513 BH Hague, The Netherlands