

Amyloid Stain - Tissue Control Slides

Description: Human.
Natural disease state (where applicable).
Paraffin embedded.

Availability: TCS0003-5 5 unstained slides.
TCS0003-25 25 unstained slides.

Storage: 2-25° C.

Suggested Stain Kit: AMY-1

Procedure: See below.

Amyloid Stain Kit (Congo Red)


Description: The Amyloid Stain Kit (Congo Red) is intended for use in the histological visualization of amyloid in tissue sections. Examination under a polarizing microscope results in green birefringence of amyloid.


Amyloid:	Red to Pink
Erythrocytes:	Light Orange
Eosinophil Granules:	Orange to Red
Nuclei:	Blue


Uses/Limitations: Not to be taken internally.
For In-Vitro Diagnostic use only.
Histological applications.
Do not use if reagents become cloudy.
Do not use past expiration date.
Use caution when handling reagents.
Non-Sterile.

Control Tissue: Freshly cut sections containing amyloid.
Cut sections 6-12 microns to show smaller amyloid deposits.


Ordering information regarding individual components on back page!

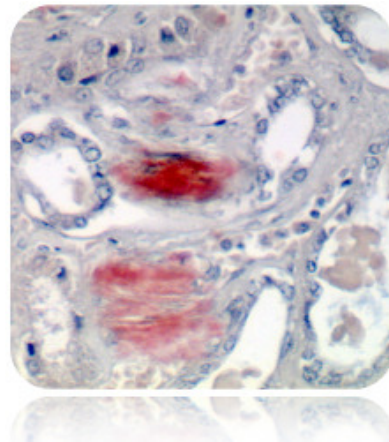

Storage: 2° C  25° C


 ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
U.S.A.


CE 

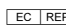 EmergoEurope (31)(0) 70 345-8570
Molsnstraat 15
2513 BH Hague, The Netherlands

Bulk Reagent Ordering Information and Current Pricing at www.scytek.com


Description:	Catalog #	Volume
Congo Red Solution	CRA500	500 ml
	CRA999	1000 ml
Hematoxylin, Mayer's (Lillie's)	HMM500	500 ml
	HMM999	1000 ml
	HMM3800	1 Gallon
Bluing Reagent	BRT500	500 ml
	BRT999	1000 ml
	BRT3800	1 Gallon

Storage: 2° C  25° C



ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
U.S.A.

 EmergoEurope (31)(0) 70 345-8570
Molsnstraat 15
2513 BH Hague, The Netherlands